

LIMITED 90 DAY WARRANTY

This Auto Power Cord is warranted for 90 days under normal use and service. This warranty extends only to customers and does not extend to retailers.

THIS WARRANTY EXPRESSLY EXCLUDES ANY AND ALL IMPLIED WARRANTIES OF MERCHANTABILITY. THERE ARE NO WARRANTIES THAT EXTEND BEYOND THE DESCRIPTION ON THE FACE HEREOF. THIS WARRANTY ALSO EXCLUDES ANY AND ALL IMPLIED WARRANTIES OF FITNESS.

To obtain warranty service on your Auto Power Cord, email us at: info@therma-zone.com or call 1-440-646-1286 ext. 1 for further instructions.

Innovative Medical Equipment does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer, or remote purchasers, to obligate Innovative Medical Equipment, LLC in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to your Auto Power Cord; improper installation; unauthorized repairs or modifications; improper use of your Auto Power Cord, ThermaZone unit or automobile; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Innovative Medical Equipment, LLC.

YOU SHOULD NOT ATTEMPT TO FIX THE AUTO POWER CORD YOURSELF OR ALLOW SOMEONE TO FIX IT. ATTEMPTING TO REPAIR THE AUTO POWER CORD YOURSELF WILL RESULT IN INVALIDATION OF THE WARRANTY.

These materials are the property of Innovative Medical Equipment, LLC

103-01 rev1 4-18-2017



AUTO POWER CORD: Instructions & Warnings



UNDER NO CIRCUMSTANCES SHOULD THE OPERATOR OF AN AUTOMOBILE OR ANY OTHER TYPE OF MOTOR VEHICLE USE THERMAZONE WHILE OPERATING THE VEHICLE.

The ThermaZone Auto Power Cord allows the ThermaZone unit to be used in any motor vehicle equipped with a 12 Volt DC power outlet.

To connect your ThermaZone unit to your motor vehicle:

1. Position your ThermaZone unit on a sturdy, flat surface inside the automobile.
2. Remove the cover from the motor vehicle's 12v accessory power outlet receptacle.
3. Plug the Auto Power Cord's small barrel connector into the receptacle on the side of the ThermaZone unit.
4. Plug the power adapter plug end of the Auto Power Cord into the 12v accessory power outlet receptacle of the motor vehicle.
5. Turn the motor vehicle's ignition to "ON" or "Accessory".
6. Verify that the green back-light on the ThermaZone unit display panel is illuminated. If not, check steps 1-5 above and make sure that the Auto Power Cord's small barrel connector and 12v accessory power outlet plug are firmly inserted into the receptacles.

CAUTION

When you are done using your ThermaZone unit, be sure to turn it off before turning off your motor vehicle. Failure to do so may cause damage to your ThermaZone unit and Auto Power Cord.



WARNING

Avoid these potentially hazardous situations which, if not avoided may result in death or serious injury:

- UNDER NO CIRCUMSTANCES SHOULD THE OPERATOR OF AN AUTOMOBILE OR ANY OTHER TYPE OF MOTOR VEHICLE USE THERMAZONE WHILE OPERATING THE VEHICLE.
- Read all instructions in the ThermaZone Instruction Manual carefully before using the ThermaZone Thermal Therapy Device.
- ONLY use your ThermaZone unit with the Auto Power Cord provided and DO NOT use your ThermaZone unit with any other DC adapter.
- DO NOT use the DC adapter with any product other than ThermaZone, and DO NOT use a DC Splitter.
- DO NOT operate in a wet or moist condition.
- DO NOT use the Auto Power Cord if it falls into water.
- Ensure close supervision when the ThermaZone unit or Auto Power Cord is being used near children or infirm persons.
- DO NOT place on or near a hot gas flame, flammable products, or electric element.
- DO NOT operate if the Auto Power Cord appears damaged or if you have dropped or damaged the device.
- DO NOT allow the ThermaZone unit or Auto Power Cord to touch hot surfaces.
- DO NOT run the ThermaZone unit or Auto Power Cord under carpeting or floor mats, or cover either with runners, or the like.
- Arrange the ThermaZone unit and Auto Power Cord away from traffic areas and where it will not be tripped over.
- Turn off your ThermaZone unit before unplugging the Auto Power Cord from the automobile outlet.
- Be sure to disconnect your ThermaZone unit from the motor vehicle's outlet when filling or emptying it with water.
- DO NOT pull or carry your ThermaZone unit by the Auto Power Cord.
- DO NOT unplug the Auto Power Cord by pulling it by the wire.
- DO NOT leave your ThermaZone unit or the Auto Power Cord in a car in which temperatures are more than 75 degrees Fahrenheit or exposed directly to sunlight.
- DO NOT leave your ThermaZone unit or the Auto Power Cord in a car in which temperatures are lower than 33 degrees Fahrenheit.
- DO NOT leave the Auto Power Cord plugged into a cigarette lighter or other power source when not in use.
- NEVER use or plug the Auto Power Cord into the cigarette lighter receptacle when jump-starting the automobile.
- The Auto Power Cord contains chemicals, including lead, known to the State of California to cause birth defects or other reproductive harm. WASH YOUR HANDS AFTER HANDLING.
- DO NOT attempt to disassemble or repair the ThermaZone unit or Auto Power Cord on your own, as that may cause a risk of electric shock or injury.

If your ThermaZone unit or Auto Power Cord needs repair, email us at info@therma-zone.com or call 1-440-646-1286 ext. 1 for further instructions.